


BUSTER Othaematoma Pad

Compression pad for treatment of othaematoma. The pad provides compression and ensures good ventilation. It helps distribute pressure evenly on the othaematoma after it has been drained for blood and fluids.

Instructions for use:

- Cut the pad to the required size with scissors
- Place the pad with the foam layer against the ear flap and the woven mesh fabric side upwards
- Before bandaging, the pinna should be positioned on the top of the dog's head
- Use a BUSTER Dog Collar or a BUSTER Ear Cover to keep the pad in place for approx. 10 days

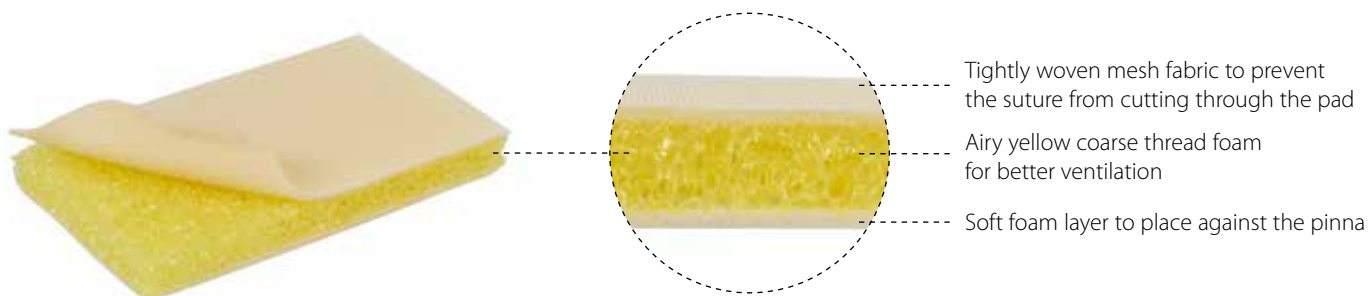

Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
161640	BUSTER Othaematoma Pad	Sterile	12 x 8 cm	4 3/4" x 3"	10/pk